



CEM SE Installation Procedures (USB Hardware Key Version)



CEM SE Installation Procedures (USB Hardware Key Version)

- These instructions cover installation on various Windows operating systems.
- The steps shown were performed using Windows 8.
- For Windows operating systems other than Windows 8, the screens may look slightly different, but the basic steps are the same.



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 1

Install all files provided with your Installation Package (CD, Cloud Based Download, etc.) to your computer.



CEM SE Installation Procedures (USB Hardware Key Version)

Step 2

Ensure that your supplied USB Hardware Key (dongle) is **NOT** installed in your computer at this point.



CEM SE Installation Procedures (USB Hardware Key Version)

Step 3

- Have the CEM SE Product Serial Number supplied with your installation package ready.
- Install CEM software as follows.....



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 4

Start the setup application file that came with your installation package.

<input type="checkbox"/> Name	Date modified	Type	Size
Acrobat	9/27/2016 12:25 PM	File folder	
CEMInstall	9/27/2016 12:25 PM	File folder	
IE55Install	9/27/2016 12:25 PM	File folder	
NTSP5	9/27/2016 12:25 PM	File folder	
SBAS	9/27/2016 12:25 PM	File folder	
SentinelUtilityInstall	9/27/2016 12:25 PM	File folder	
AUTORUN.INF	9/27/2016 12:25 PM	Setup Information	1 KB
CEM.ico	9/27/2016 12:25 PM	ICO File	1 KB
setup.exe	9/27/2016 12:25 PM	Application	1,016 KB




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 5

Click the “Install CEM” button to launch the CEM Installation Wizard.

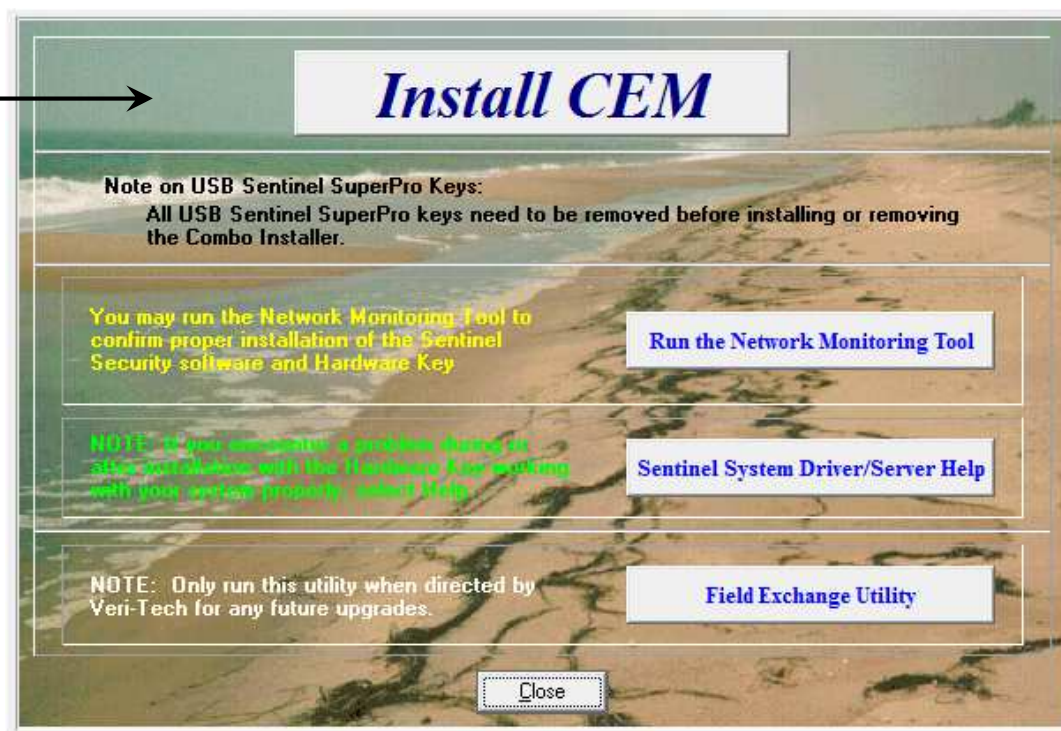



CEM SE Installation Procedures (USB Hardware Key Version)

Step 6

Review info and click “Next”
when ready.






CEM SE Installation Procedures

(USB Hardware Key Version)

Step 7

Review info and click “Yes” when ready.

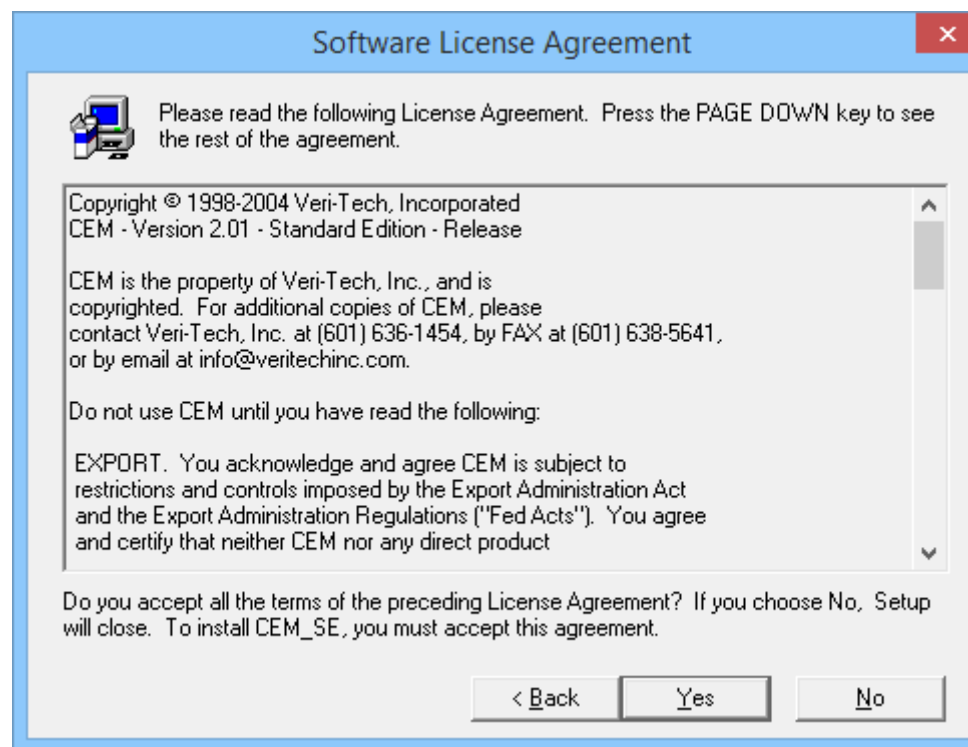




CEM SE Installation Procedures (USB Hardware Key Version)

Step 8

Enter your Name and Company info, and the CEM Product Serial Number.

Click "Next" when ready.

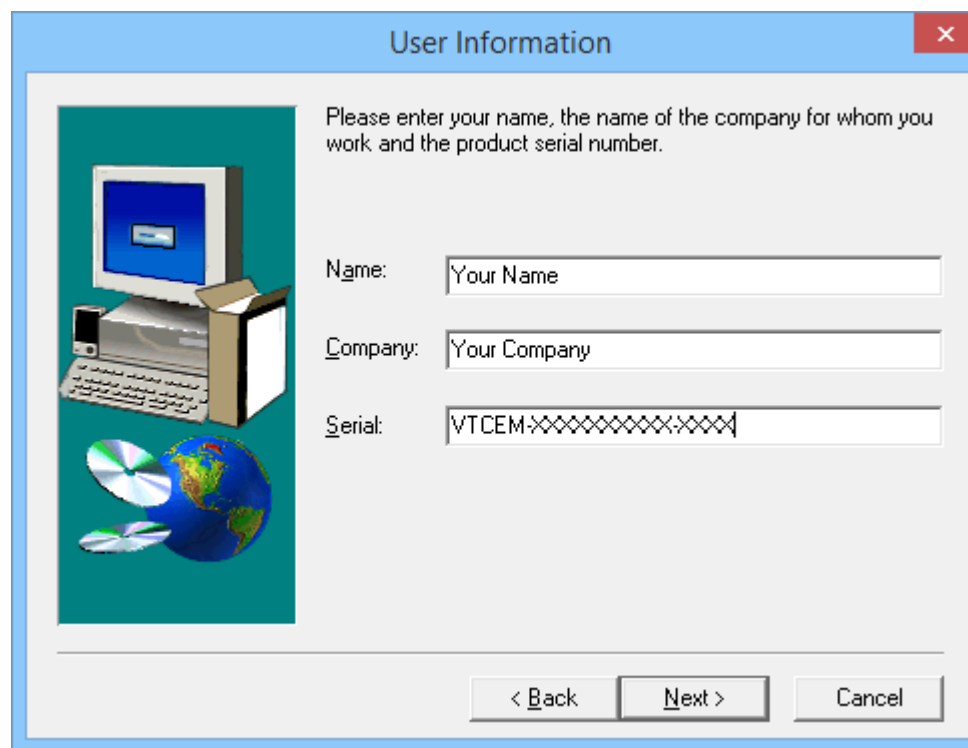

The image shows a Windows-style dialog box titled "User Information" with a red close button in the top right corner. The dialog has a light blue border. On the left side, there is a graphic showing a computer monitor, keyboard, and mouse, with a globe and two CD-ROMs below it. To the right of the graphic, there is a text prompt: "Please enter your name, the name of the company for whom you work and the product serial number." Below this prompt are three text input fields. The first is labeled "Name:" and contains the text "Your Name". The second is labeled "Company:" and contains the text "Your Company". The third is labeled "Serial:" and contains the text "VTCEM-XXXXXXXXXXXX". At the bottom of the dialog, there are three buttons: "< Back", "Next >", and "Cancel".




CEM SE Installation Procedures (USB Hardware Key Version)

Step 9

The Installation Wizard creates a Default Destination Folder.

Even though an alternate Destination Folder can be created, it is recommended that the default location be used.

Click “Next” when ready.

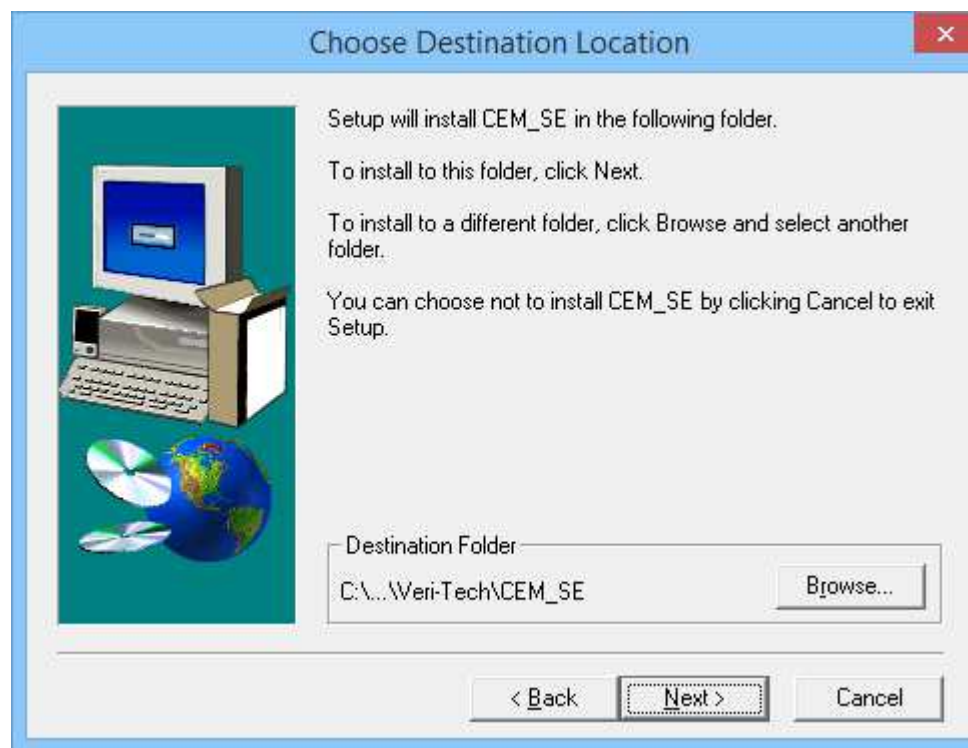




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 10

Review info and click “Yes”
when ready.

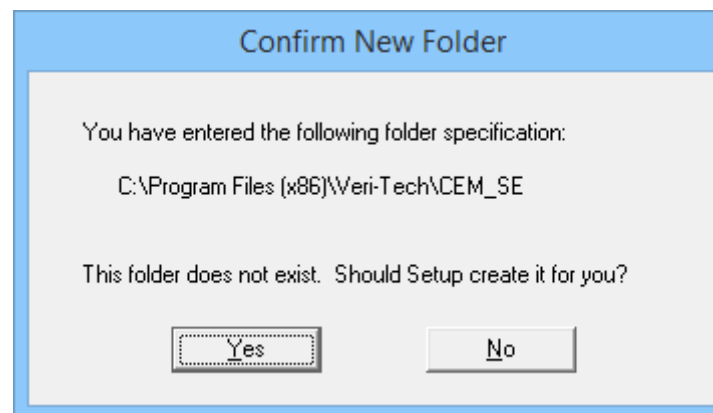




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 11

Review info and click “OK”
when ready.

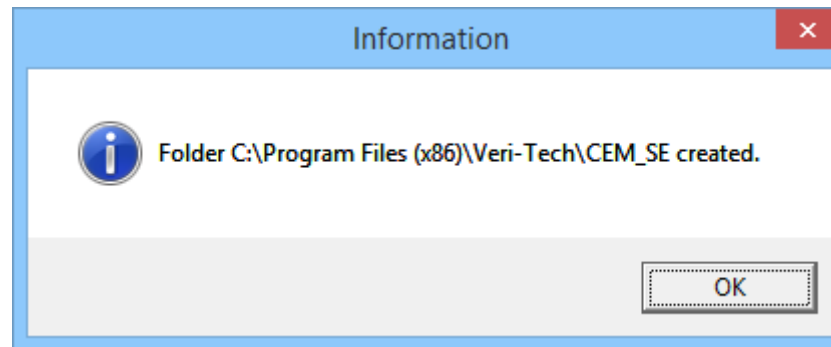




CEM SE Installation Procedures (USB Hardware Key Version)

Step 12

Review info and click “Next” when ready.

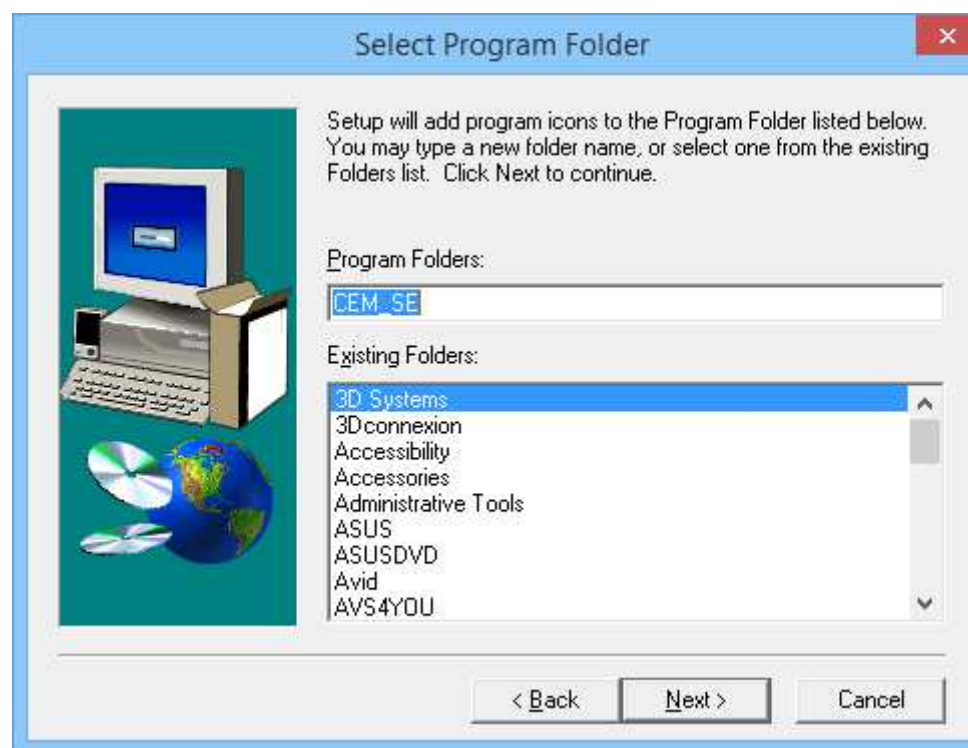




CEM SE Installation Procedures (USB Hardware Key Version)

Step 13

Review info and click “Next”
when ready.

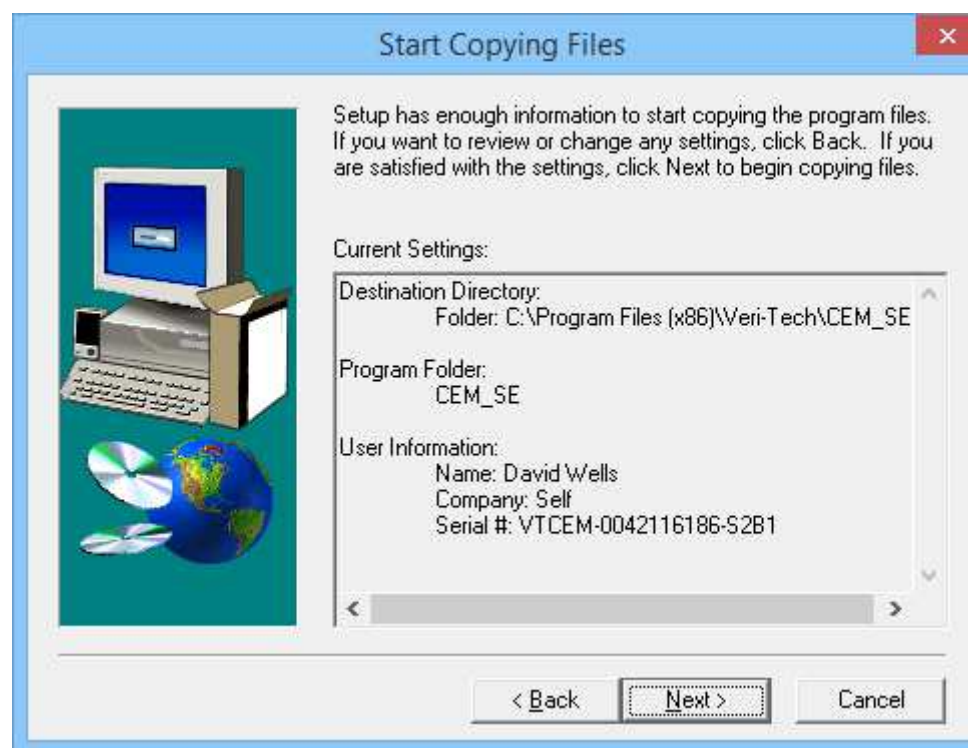




CEM SE Installation Procedures (USB Hardware Key Version)

Step 14

This error message
is normal.
Click "OK".

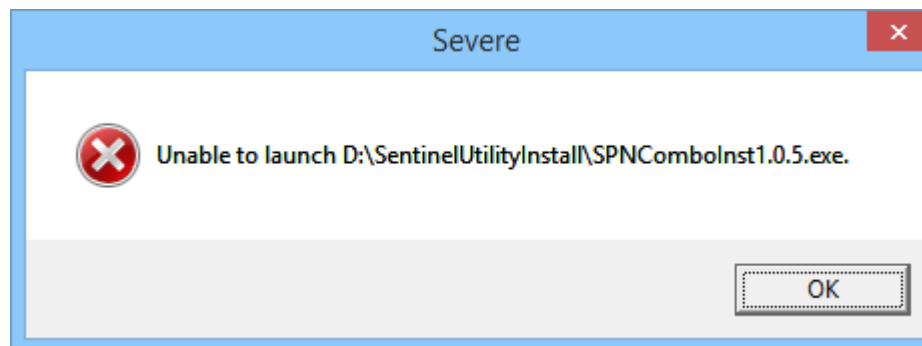




CEM SE Installation Procedures (USB Hardware Key Version)

Step 15

Review info and click “Finish”
when ready.

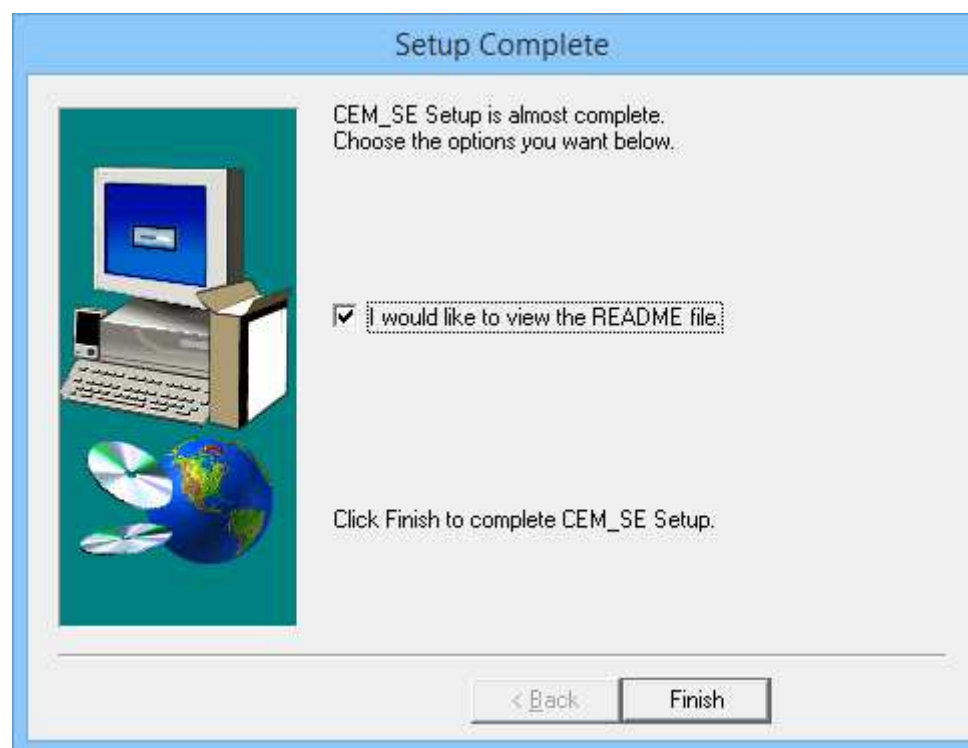



CEM SE Installation Procedures (USB Hardware Key Version)

Step 16

CEM SE program is now installed, and a Short Cut created on your Desktop.




The Hardware Key will be installed next.



CEM SE Installation Procedures (USB Hardware Key Version)

Step 17

Leave the CEM Installation Utility Window open for later use.

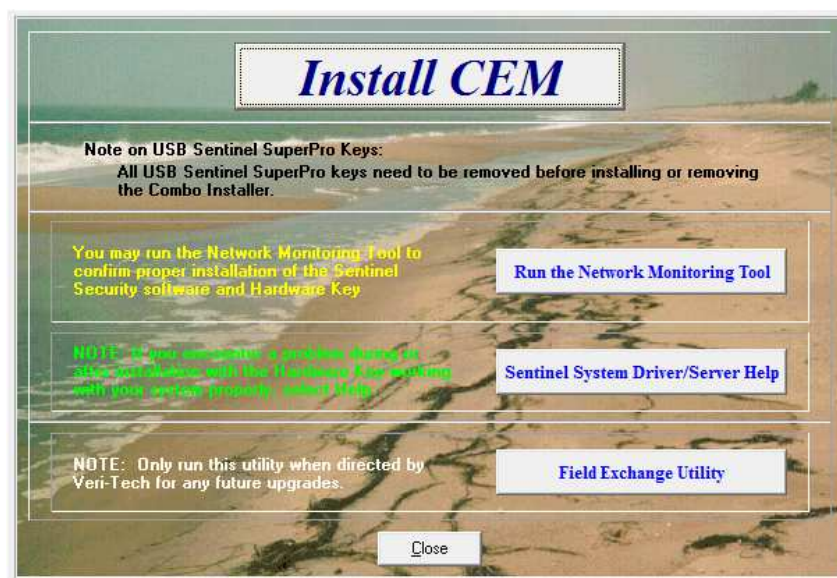



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 18

For Windows XP, VISTA, 7, 8 and 10 Operating Systems only

- Download the latest Sentinel Key Driver (currently v7.6.8) from the following link:

https://supportportal.gemalto.com/csm/?id=kb_article&sys_id=eb25edfd37890b80cc47261953990e60

- This link will be provided with your installation package.

For other legacy Windows Operating Systems

- Download the Sentinel Key Driver version 7.6.0 from the following link:

https://supportportal.gemalto.com/csm/?id=kb_article&sys_id=e88691ba37edcb08cc47261953990e80



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 19

(Steps 19 – 27 are shown for installation of version 7.6.8. For Version 7.6.0 the screens may look slightly different, but the basic steps are the same)

Download and Extract the Driver files. Make sure to allow pop-ups in your browser.

The screenshot shows a Gemalto article page for 'Sentinel Protection Installer 7.6.8 for Windows'. The page includes the Gemalto logo, the article title, a KB number (KB0016363), and an 'Article Details' section with 145 views and a date of 2017-09-07. The 'Components' section lists: Sentinel System Driver 7.5.9, Sentinel Security Runtime 1.0.1, Sentinel Protection Server 7.6.4.0, and Sentinel Keys Server 1.3.5.0. The 'Platforms Supported' section lists various Windows operating systems from XP to Windows Server 2012. A note at the bottom states that the installer does not support Windows 10 Professional with 'Device Guard' enabled. A link at the bottom right says 'Click here to download file (browser pop-ups must be allowed): [DOW0002231](#)'. An arrow from the text 'Make sure to allow pop-ups in your browser.' points to this link.



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 20

Recommended to extract these files to a newly created folder, located with the other CEM SE installation files.


Name	Date modified	Type	Size
Acrobat	9/27/2016 12:25 PM	File folder	
CEMInstall	9/27/2016 12:25 PM	File folder	
IE55Install	9/27/2016 12:25 PM	File folder	
NTSP5	9/27/2016 12:25 PM	File folder	
SBAS	9/27/2016 12:25 PM	File folder	
Sentinel Key Driver	9/28/2016 5:50 PM	File folder	
SentinelUtilityInstall	9/27/2016 12:25 PM	File folder	
AUTORUN.INF	9/27/2016 12:25 PM	Setup Information	1 KB
CEM.ico	9/27/2016 12:25 PM	ICO File	1 KB
setup.exe	9/27/2016 12:25 PM	Application	1,016 KB





CEM SE Installation Procedures (USB Hardware Key Version)

Step 21

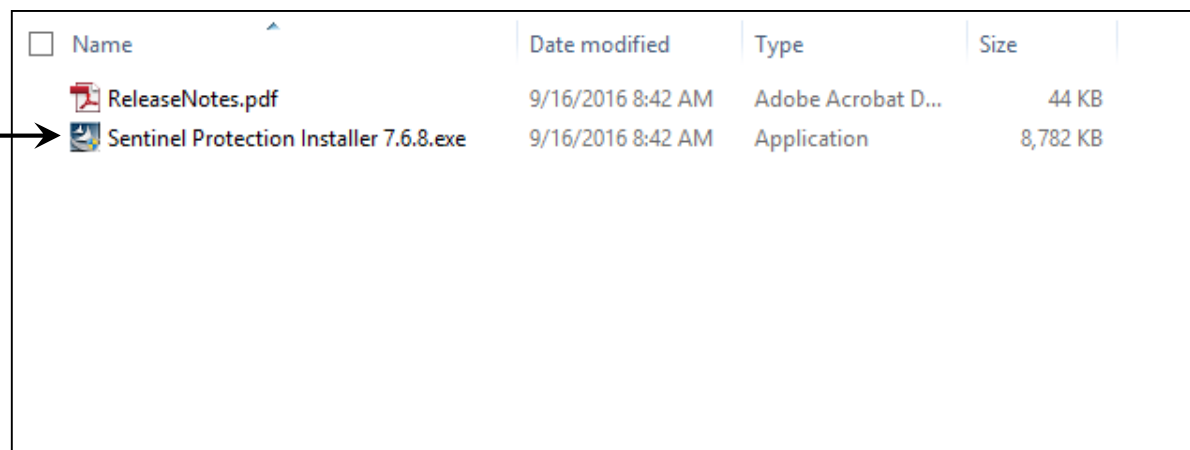Launch the
Sentinel Key
Installation
Wizard



<input type="checkbox"/> Name	Date modified	Type	Size
 ReleaseNotes.pdf	9/16/2016 8:42 AM	Adobe Acrobat D...	44 KB
 Sentinel Protection Installer 7.6.8.exe	9/16/2016 8:42 AM	Application	8,782 KB



CEM SE Installation Procedures (USB Hardware Key Version)

Step 22

Review info and click “Next” when ready.





CEM SE Installation Procedures

(USB Hardware Key Version)

Step 23

Review info and click “Next” when ready.






CEM SE Installation Procedures

(USB Hardware Key Version)

Step 24

Review info and click “Next” when ready.

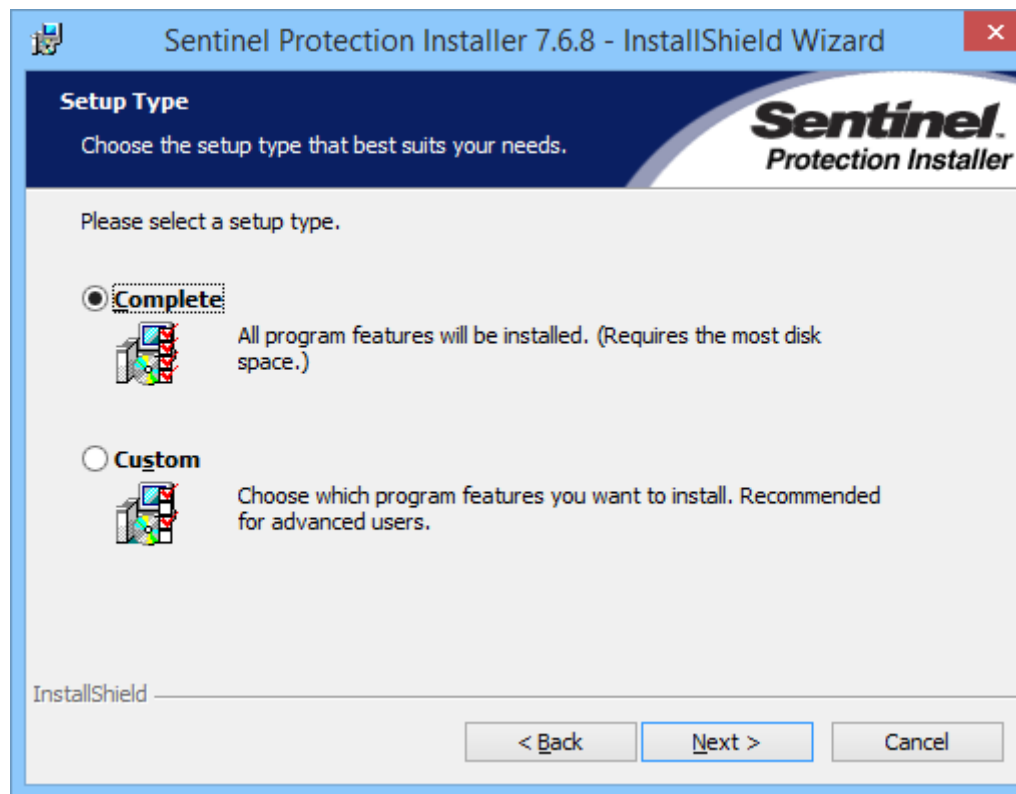




CEM SE Installation Procedures (USB Hardware Key Version)

Step 25

Review info and click “Install”
when ready.

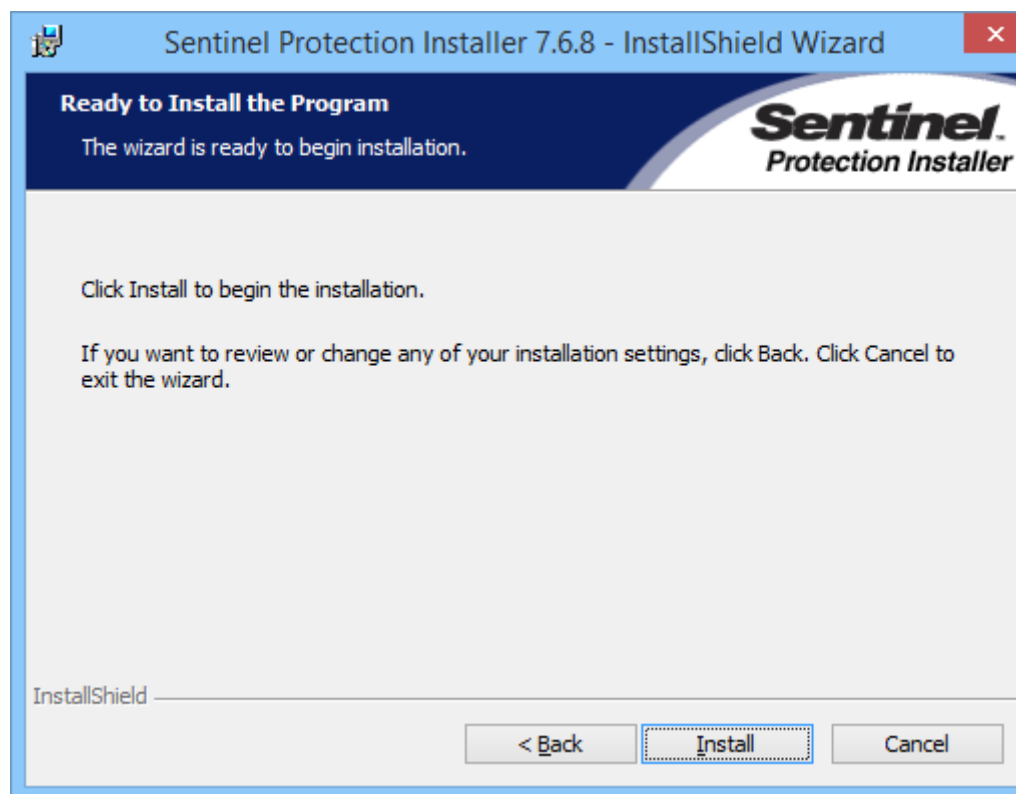




CEM SE Installation Procedures (USB Hardware Key Version)

Step 26

Review info and click “Yes”
when ready.

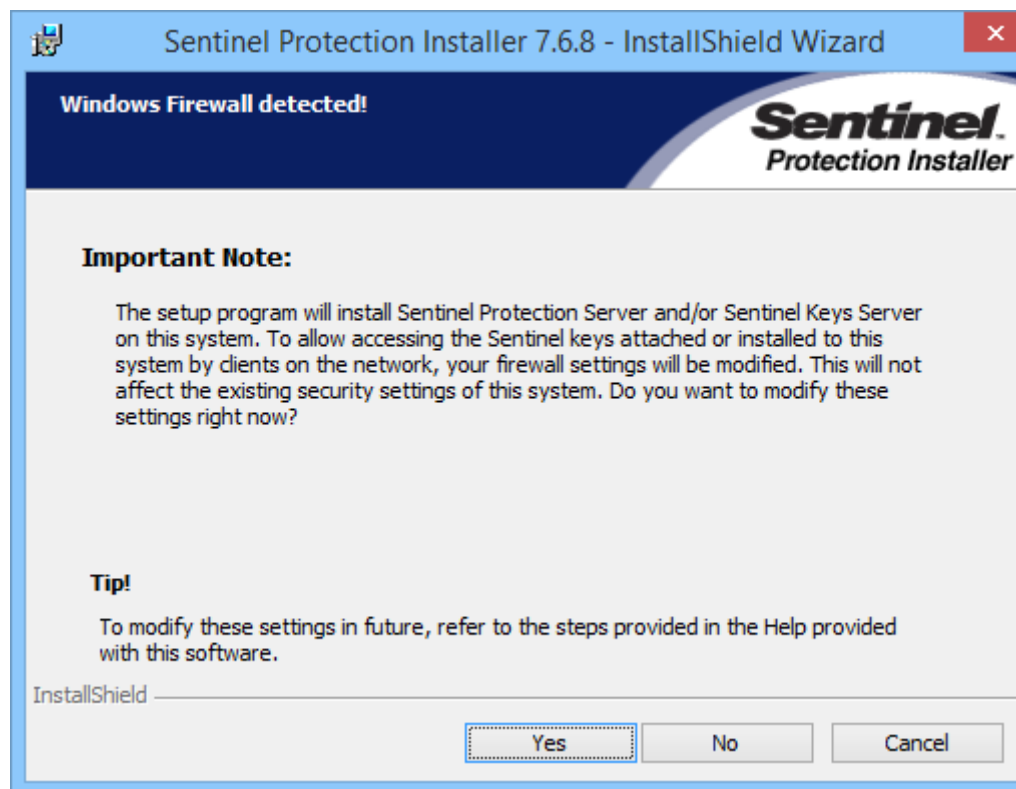




CEM SE Installation Procedures (USB Hardware Key Version)

Step 27

Review info and click “Finish”
when ready.

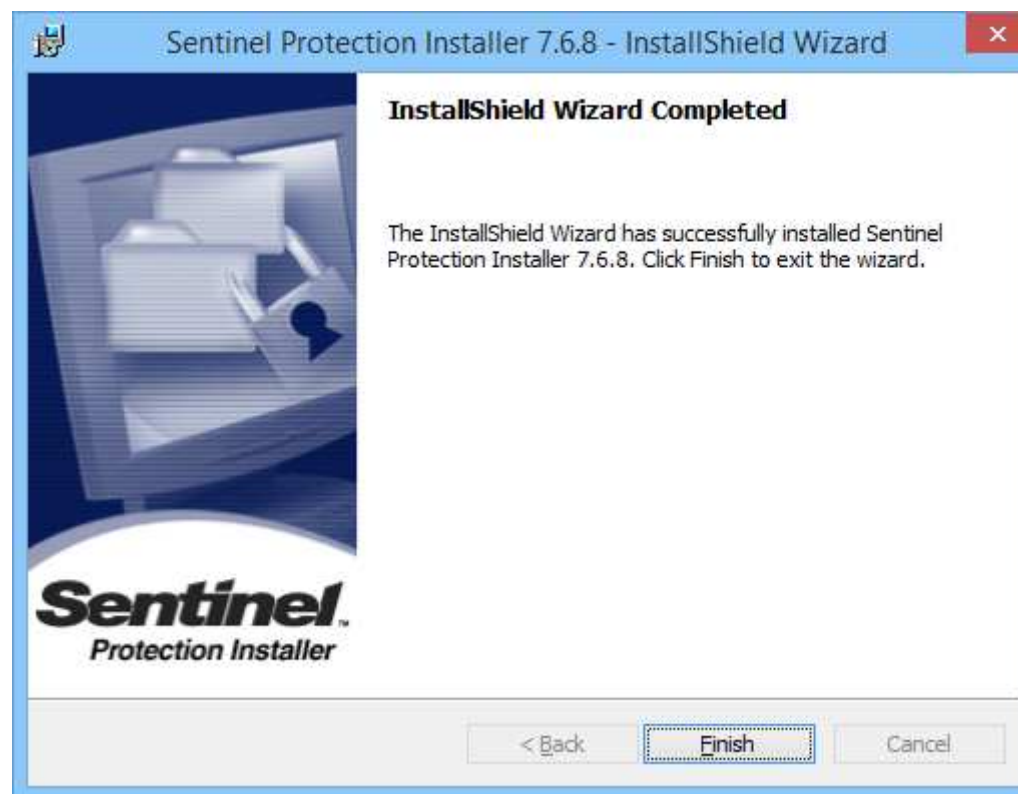



CEM SE Installation Procedures (USB Hardware Key Version)

Step 28

At this point, you can install the supplied USB Hardware Key into your computer.




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 29

Click the “Run the Network Monitoring Tool” button.

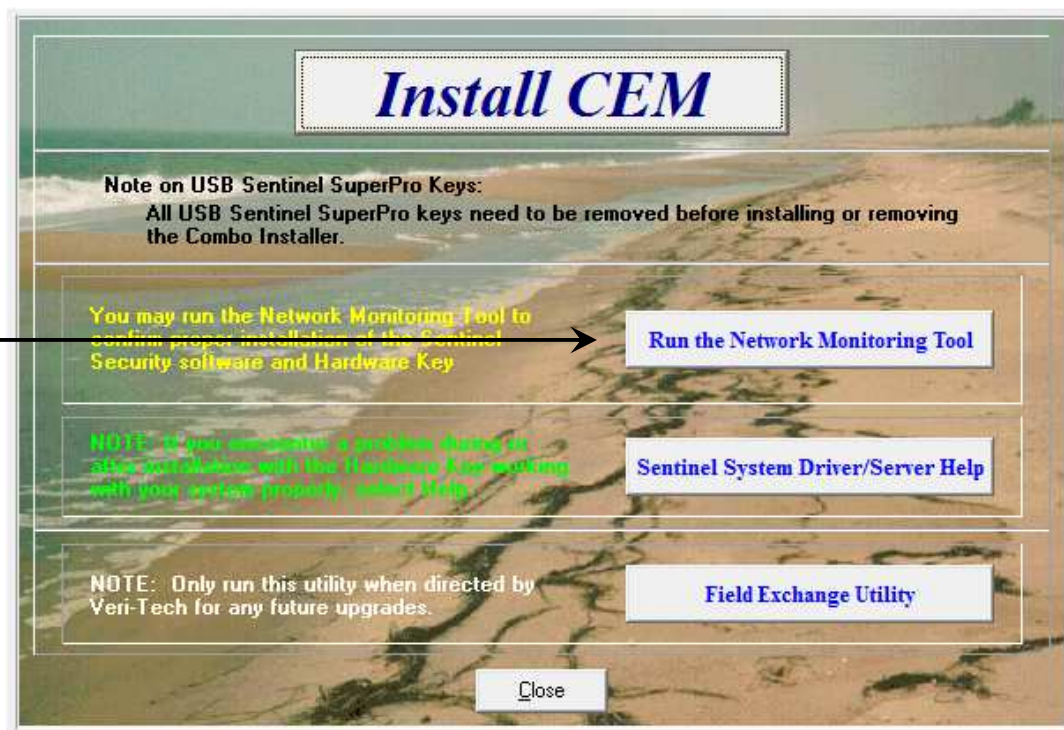




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 30

Make sure the “TCP/IP” radio button is selected, and click the “OK” button.

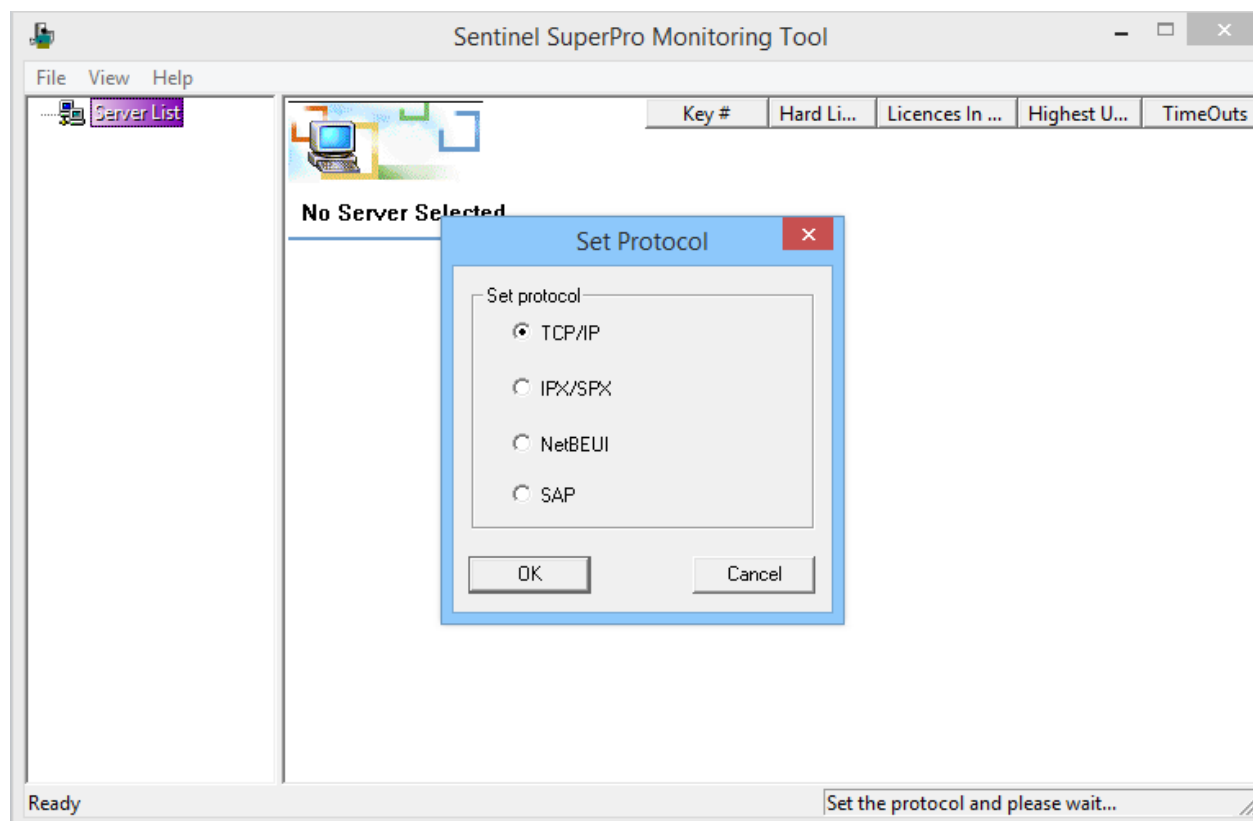




CEM SE Installation Procedures

(USB Hardware Key Version)

Step 31

The Monitoring Tool verifies that the USB Hardware Key and Driver are correctly installed.

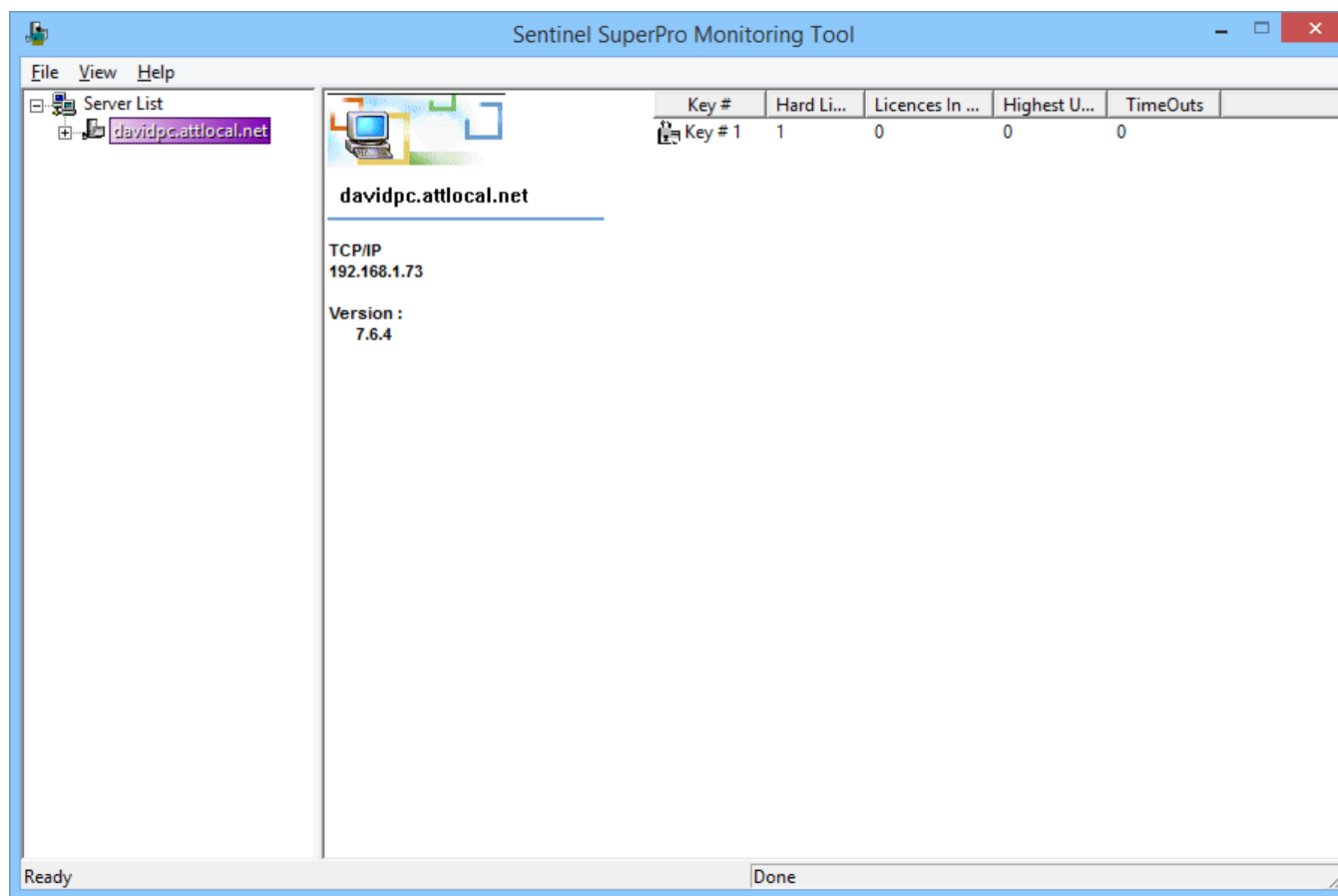



CEM SE Installation Procedures

(USB Hardware Key Version)

Step 32

Launch CEM SE from
the previously installed
Desktop Shortcut. →





CEM SE Installation Procedures

(USB Hardware Key Version)

Step 33

CEM SE installation is now complete.

